How to self-collect a urine sample

Read the collection instructions. If you have any questions about how to collect the sample, please ask your healthcare provider.

Preparing for sample collection:

- Wash hands prior to collection.

Please follow these steps to assure reliable results and reduce the need for retesting.

Note:

- Patient should not urinate 1 hour prior to sampling.
- It may be helpful to mark the volume required on the cup.
- For best results, female patients should not cleanse the labial area prior to collection.

Collecting male/female urine

1.

PREPARE: Position the urine cup to collect the beginning of the urine stream.

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2.

COLLECT URINE: Begin urinating. Collect the first 10-50 mL of urine.

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3.

SECURE CONTAINER: Close the lid of the urine cup securely, label and date.

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How to transport samples for testing

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Handling precautions:

The collection tube media can cause irritation if contacted with skin or other body parts. Handle the collection tube carefully.

- Use care to avoid splashing contents of the tube. If the contents of the tube are spilled on your skin, wash the affected area with soap and water. If the contents of the tube splash into your eyes, flush them with water immediately. Always notify your healthcare provider. NOTE: In case the contents of the tube are accidentally spilled, do not attempt to clean up. Immediately notify your healthcare provider for appropriate action. If the urine specimen cannot be transferred immediately, it can be stored at 2°C to 30°C for up to 24 hours.

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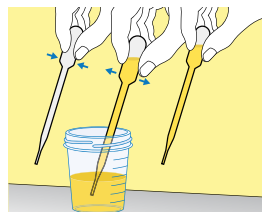
How to transfer urine specimen to cobas PCR tube

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
- 1. COLLECT:** Prior to sampling, the patient should not have urinated for at least one hour. Given that collection of larger volumes of urine may reduce test sensitivity, please direct patient to provide first-catch urine (approximately 10 to 50 mL of the initial urine stream) into a urine collection cup (not provided).

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
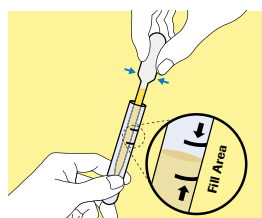
- 3. TRANSFER:** The correct volume of urine has been added when the fluid level is between the two black lines on the tube label.

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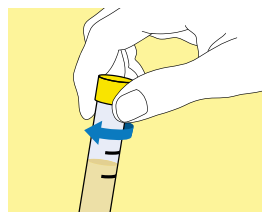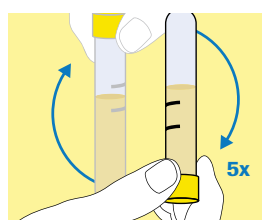cobas[®] PCR Media tube: Shipping and storage

- Urine specimens collected in **cobas**[®] PCR Media are stable at 2 - 30°C for up to 1 year.

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